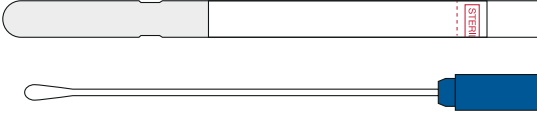
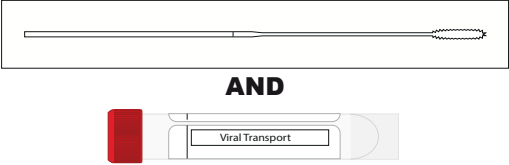
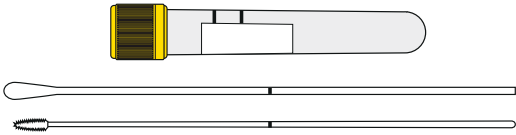
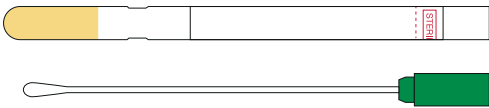
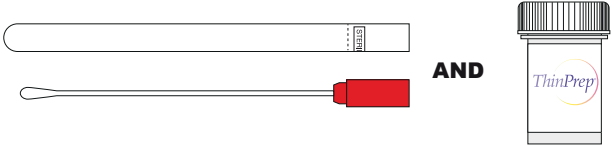


Swab Identification

	Swab Description	Use	Instructions
MICROBIOLOGY	<p>Bacteriology Transport Swab (Blue Top) Contains Amies Transport Medium</p> 	<ul style="list-style-type: none"> Bacterial culture from any site To isolate bacteria, fastidious organism and yeasts For screening: <ul style="list-style-type: none"> - MRSA - VRE - Multi-resistant gram negative bacteria (MRGN) Includes <i>Bacterial Vaginosis</i> 	<p>May be left overnight at room temperature except where anaerobic infection is suspected.</p> <p>Bacterial Culture only - NOT for PCR or SYPHILIS.</p>
PCR	<p>Liquid swab (Swab Nasal Samp Kit 10ML TBE 3ML MEDIUM50 (VIRAL))</p> 	<ul style="list-style-type: none"> PCR for: <ul style="list-style-type: none"> - Enteroviruses - <i>Bordetella pertussis</i> - Respiratory viruses - <i>Mycoplasma pneumoniae</i> and/or other causes of atypical bacterial pneumonia 	<p>Store at room temperature before and after collection.</p> <p>Please note: swab collected for viral testing should not be placed in Amies Transport Media.</p> <p>NOT suitable for bacterial culture.</p>
PCR	<p>COBAS PCR Media Dual Swab</p> 	<ul style="list-style-type: none"> Molecular testing (genital, rectal, throat and eye): <ul style="list-style-type: none"> - <i>Chlamydia trachomatis</i> - <i>Neisseria gonorrhoeae</i> - <i>Mycoplasma genitalium</i> - <i>Trichomonas vaginalis</i> 	<p>No special handling or transport requirements.</p> <p>Please note: Swab selection will vary depending on the site from which sample is to be collected.</p> <p>NOT suitable for bacterial culture.</p>
PCR	<p>Viral Transport Swab (Green Top) or Dry Swab Contains viral transport media (VTM)</p> 	<ul style="list-style-type: none"> Viral PCR for: <ul style="list-style-type: none"> - CMV - HSV, VZV - Enteroviruses - Other viruses e.g. measles 	<p>Store at room temperature before and after collection.</p> <p>NOT suitable for bacterial culture.</p>
CYTOLOGY-HPV	<p>Cervical Screening Self Collect Kit</p> 	<ul style="list-style-type: none"> HPV PCR ONLY <p>Note: This swab is only used in the Self Collect Kit.</p> <p>Self-collect - HPV Specimen Collection. Vaginal samples only.</p>	<p>A Self Collect HPV Specimen guide is available at lavery.com.au in the Cervical screening section.</p> <p>Samples submitted without swirling into a Thinprep vial cannot be processed at Lavery.</p>